

OWNERS GUIDE

Cenotaph Products LLC

MODEL NO. CCV2.6

October 2019

CONTENTS

Cradle Unit	Owner's Manual	Thermal Blanket
Power Cord	Quick Start Guide	Unit Key
Cooling Mattress Pad		

CRADLE SPECIFICATIONS

- 2 Year Warranty
- 5 Year Warranty on Compressor and Sealed System
- Caring Cradle Dimensions: 42" L X 21" W X 42.5" H
- 110 Volt AC Converts to 24 Volts DC, 15 Amp Circuit Required
- Smart Cradle can run on voltages worldwide. 110/220 Volts
- Unit Weight: Approximately 110 - 120 pounds
- Refrigerant: R134a Tetrafluoroethene Freon
- Operating Temp Range: (45°F +/- 3°F)
- Exterior Temp Operating Range: (64°F - 75°F)
- Cradle is manufactured in the United States.
- Certified Medical Grade Electronics Installed in this Device
- Bassinet Material: Clear Polycarbonate with rigid construction
- Internal keyed locking system (2 provided) for added security.
- 4 in. x 2 in. easy glide casters provide effortless mobility all locking

Compliance: This device complies or exceeds with hospital equipment standards for use within hospital environments.
UL 61010-1:2012 CAN/CSA-C22.2 NO. 61010-1-12



Why This Device Was Invented

This invention relates to an improved and dignified means for parents and families to spend time with a baby who has died before, during or after birth. This "gift of time" allows memory making, keepsake creating, photos to be taken as well as others to meet the baby before the natural changes that occur in death alter the infant's appearance. These changes include but are not limited to, discoloration, lividity, skin peeling, odor, and fluid leaking. This invention is intended to accommodate any pregnancy loss or infant death from 20 weeks gestation thru 1 year of age.

CENOTAPH PRODUCTS LLC

- Multiple Patents Pending.
- Multiple Utility & Design Patents Pending
- Designed, Engineered, & Manufactured in the USA
- Contact info:

Phone: +1 586-935-2022

Fax: +1 615-501-9661

info@cenotaphproducts.com

www.cenotaphproducts.com

OVERVIEW

The Cenotaph Amish Cradle is a cooling unit designed and engineered to allow parents and families the ability to say goodbye to their baby in a private comfortable atmosphere.

WARNING!

BE SURE TO READ ALL PRODUCT SAFETY, SETUP AND OPERATIONAL INSTRUCTIONS IN THIS GUIDE BEFORE USING THE SMART CRADLE DEVICE. PROPER SETUP, INSTALLATION, AND USAGE AS OUTLINED IN THIS OWNER'S MANUAL ARE ESSENTIAL FOR THE CRADLE TO FUNCTION PROPERLY. THIS OWNER'S MANUAL CONTAINS IMPORTANT SAFETY AND PRODUCT INFORMATION THAT IDENTIFIES HAZARDS THAT COULD RESULT IN PERSONAL INJURY, DEATH AND/OR PROPERTY DAMAGE. PAY CLOSE ATTENTION AND THOROUGHLY READ THIS OWNER'S MANUAL.

PRODUCT SAFETY

DO NOT DISCARD. KEEP THIS GUIDE FOR REFERENCE.

KEEP OUT OF CHILDREN'S REACH.



Why the name Cenotaph?

By definition:

A cenotaph is a monument erected in honor of a person or group of people whose remains are elsewhere. It can also be the initial tomb for a person who has since been reinterred elsewhere. Since the vast majority of cenotaphs honor individuals, we purposely chose this name to honor the babies whose parents have chosen to remember the lost in a befitting way and dedicate this device to helping others going through this extremely hard process giving these families the time to spend with their baby. This helps aid in the healing process of the parents, grandparents, everyone involved.

Therefore, it is our goal is to supply anyone the ability to purchase a Cenotaph Cradle at wholesale pricing regardless of their affiliates. If you belong to a nonprofit great. If you're wanting to purchase one and gift it to a hospital or birthing center that's great as well.

Differences between the Cenotaph Cradle and the competition:

1. Pricing – our goal is to supply anyone the ability to purchase a Cenotaph Cradle at wholesale pricing regardless of their affiliates. If you belong to a nonprofit great. If you're just wanting to purchase one and gift it to a hospital or birthing center that's great as well.
2. Designed, Engineered, & Assembled in the USA.
4. Amish Cradle is built in a medical device manufacturing facility located in the USA.
5. **Multiple Utility and Design Patents Pending on this device.**
6. **2 Year Warranty. Please see Warranty Section.**
7. **5 Year Warranty on Compressor and Sealed System**
8. **No harmful chemicals to deal with.**
9. No monitoring of the unit. Plug it in, switch to on and GO.
10. Takes 15/20 minutes or less to reach the desired temperature.
11. Custom designed and engineered evaporator made from bonded rolled aluminum for maximum reliability.
12. Cradle is fine-tuned for optimum efficiency with STATE OF THE ART controls and components that have been custom designed to meet today's energy demands. There is nothing on the market today anywhere close to the efficiency of our Amish Cradle.
13. Support – Please let us know how we can support you or your group in creating something specific to your needs or wants.
14. We supply the Cradle with no markings or logos. Each group or individuals have the ability to customize their Cradle to whatever you want it to look like.
15. Let's see what we can build together?

LIMITED WARRANTY

Cenotaph Products LLC ("CP") warrants this Amish Cradle ("device") against defects in material or workmanship as follows:

PARTS & LABOR: For a period of 2 full years from the date of shipment from the manufacturer, if this Cradle is determined to be defective, CP will repair or replace it at an authorized service facility. After the 2 Year Period, parts, labor, and shipping charges will apply. If there is damage to the unit caused by the end user all parts, labor and shipping will remain the responsibility of the customer.

5 Year Limited Warranty on the sealed system and compressor. In the unlikely event of a compressor or sealed system failure on this device, CP is happy to replace defective parts at no charge. Labor and shipping will be the responsibility of the customer.

To obtain warranty service, the unit will have to be delivered freight prepaid, in either the original packaging or packaging affording an equal degree of protection, to an authorized service facility.

Please contact us if you have any questions concerning customer instructions, installation, or set up.

The Cradle by CP is warranted to be free from defects in material and workmanship under normal use. This warranty does not cover defects or deficiencies in the product to the extent that such defects or deficiencies are the results of customer improper usage.

Defective Products shall specifically exclude, without limitation, any Products (i) which have been subjected to misuse, negligence, accident, or improper maintenance, installation, or application or (ii) which have been repaired or altered without CP prior written consent or (iii) which identification marks and labels have been altered or removed.

NO OTHER WARRANTIES, EXPRESS OR IMPLIED, ARE MADE WITH RESPECT TO THE PRODUCT S INCLUDING, BUT NOT LIMITED TO, ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE.

At CP's consideration, customer shall either (i) return to a location designated by CP, any allegedly defective products for which claims are made, with a written explanation of the claimed failures, for CP's inspection. If the Defective Products are determined to be in compliance with CP warranty to the reasonable satisfaction of all involved, they shall be returned to, or retained by, customer, and customer shall bear the cost of any freight and duty in both directions and any costs incurred by CP in inspecting the Products.

IN NO EVENT, WHETHER AS A RESULT OF BREACH OF CONTRACT OR WARRANTY, ALLEGED NEGLIGENCE OR OTHERWISE, SHALL EITHER PARTY BE LIABLE TO THE OTHER FOR SPECIAL, INCIDENTAL OR CONSEQUENTIAL DAMAGES, INCLUDING BUT NOT LIMITED TO LOSS OF PROFITS OR REVENUE, LOSS OF USE OF PRODUCTS OR OTHER EQUIPMENT, OR DOWNTIME COSTS.

This warranty is valid Globally. Shipping changes on all warranties will apply outside of the United States.

This warranty is invalid if the factory applied serial number has been altered or removed from the device.

REPAIR OR REPLACEMENT AS PROVIDED UNDER THIS WARRANTY IS EXCLUSIVE REMEDY OF THE CONSUMER. CP SHALL NOT BE LIABLE FOR ANY INCIDENTAL OR CONSEQUENTIAL DAMAGES FOR BREACH OF ANY EXPRESS OR IMPLIED WARRANTY ON THIS PRODUCT EXCEPT TO THE EXTENT PROHIBITED BY APPLICABLE LAW. ANY IMPLIED WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE ON THIS PRODUCT IS LIMITED IN DURATION TO THE DURATION OF THIS WARRANTY.

To obtain service under this warranty, contact us at: info@cenotaphproducts.com or visit us at: www.cenotaphproducts.com

Date of shipment from Manufacturer:

Serial number:

OWNER'S GUIDE

PRODUCT INSTALLATION

- Remove from shipping box.
- Cradle must always remain in the upright position at all times. Unit should never be laid on its side. It is never to be transported on its side.
- Smart Cradle can run on voltages worldwide. United States Market 110 Volts, 15 amp or 220 Volt, 15 amp system found in many other countries throughout the world. Whatever voltage is delivered to the unit it will be converted to 24 Volts thus making the system quiet, dependable, and energy efficient.
- Once plugged into the wall outlet, switch to the ON position.
- The Cradle is factory preset at 45 degrees Fahrenheit, there is no need to make any adjustments to the cooling system.
- The unit operates best in an ambient room temperature of 64°F to 75°F.

OPERATIONS

- Within 20/30 minutes of activation, the device will reach a temperature of 45°F +/- 3°F depending on the ambient room temperature. Once the bottom of the bassinet and pad feels cold the unit will be ready to be used.
- The unit should be checked daily to confirm the floor of the bassinet continues to be cold to the touch.
- This device is designed to run 24 hours a day for up to 7 full days of consistent use.
- Keep baby covered with lined blanket while in the cradle to help maintain cooling.

DISCLAIMER AND LIMITATIONS OF REMEDIES

This product is not intended to diagnose, treat, cure, mitigate, or prevent disease, illness or any medical condition. This product is not to be used as a replacement for medical treatment. If you have a disease, illness or medical condition, consult with your physician or healthcare provider before using the product. A licensed medical professional should be consulted for diagnosis and treatment of any and all medical conditions. This products is neither approved nor endorsed by any governmental medical device authority or other agency. All implied warranties, including those of merchantability and fitness for a particular purpose are excluded and are disclaimed.

Use only as directed.

Contact Us

Cenotaph Products LLC

Ph: +1 586-935-2022

Fax: +1 615-501-9661

email: info@cenotaphproducts.com

www.cenotaphproducts.com

Cenotaph Products LLC Model CCV 2.6

Cenotaph Products LLC

PLACE
STAMP
HERE